

CLAVE[®] Needlefree Connector

Catalog Nos. C1000, C1000G, C1000-GR, C1000-RR, C1000-YR
For Needleless Access to Central, Peripheral, and Arterial Lines

- *More than an 18 gauge flow rate*
- *Backpressure = 60psig.*
- *Lipid, Blood, Chemo-drug compatible*
- *Residual Volume (dead space) = 0.06cc*
- *Contains no natural rubber latex.*



PRIME the CLAVE


1. Remove from package and attach IV administration set or priming syringe to female luer of CLAVE.
2. Prime CLAVE per facility protocol taking care to invert device to expel air.
3. Remove protective cap and attach male luer of CLAVE to IV extension set or catheter.

SWAB and ACCESS the CLAVE

1. Swab silicone seal of CLAVE Connector before each access with desired disinfectant per facility protocol.
2. To access with luer lock or luer slip IV device, grasp CLAVE and firmly push and twist device into CLAVE until tight. To remove device, grasp CLAVE and twist apart until loose.
NOTE: If using a rotating luer device, first secure the luer and then "lock down" rotating collar. This will ensure a secure connection and optimal flow rates.
3. The CLAVE Connector is a closed barrier to bacteria. **Do not use end caps.**

FLUSH the CLAVE

1. Flush the CLAVE Connector after each use per facility protocol.

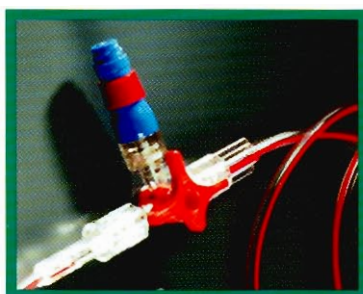

CAUTION:

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- It is recommended this device be changed per CDC guidelines or per validated facility protocol.